




PATIENT GUIDE

Caring for yourself and Brijjits post-surgery

BATHING/SHOWERING

It's ok to shower if and when advised by your physician. You can dab around with a towel, but don't rub too hard as you may displace the Brijjit(s).

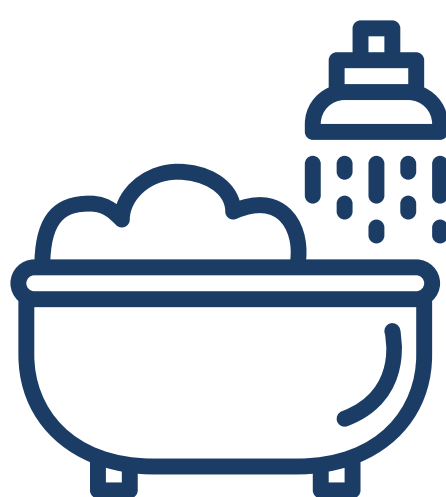

COVERING THE BRIJJIT

Do not apply adhesive bandages directly onto Brijjit. Bandages are ok as long as the sticky part does not adhere to the Brijjit itself, which could make removal of the bandage difficult.

LOTIONS/OILS

Do not apply any oil-based lotions to skin around the Brijjit as it may disrupt the adhesive.



OTHER THINGS TO KNOW

- Minor redness is ok. If you notice significant redness or irritation, please consult your physician.

If the Brijjit is displaced prematurely or your wound opens, please consult your physician.

FOR ANY OTHER QUESTIONS OR CONCERNS, PLEASE CONSULT YOUR PHYSICIAN.

Brijjit is a class 1 medical device used for non-invasive wound closure and support.
www.brijjitmedical.com




ABOUT THE PRODUCT

Offloading tension along the wound site

FORCE MODULATING TISSUE BRIDGE

Invented by a past president of the American Society for Aesthetic and Plastic Surgery, the Brijjit® Tissue Bridge is considered a breakthrough technology.

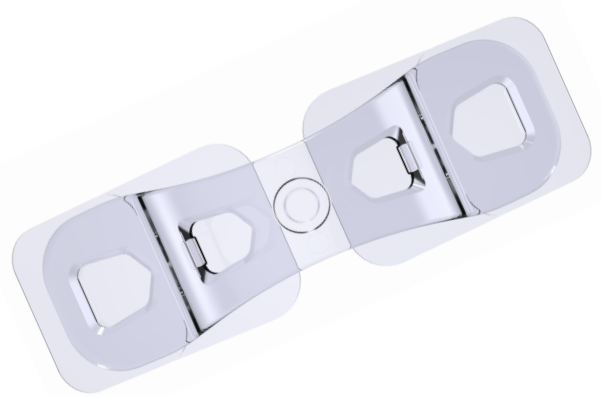

WHY BRIJJIT?

Unlike traditional sutures that cinch, pull and knot, the Brijjit device is designed to gently close the skin naturally and non-invasively, while promoting scar-friendly healing.

HEAL BEAUTIFULLY

Tension on a healing wound is one of the main factors contributing to poor scarring. Brijjit is designed to offload the tension on your wound for prolonged periods of time to help you heal beautifully.



EASY TO WEAR

Brijjit uses an adhesive common in the medical industry due to its reputation for being gentle on the skin. The bridge design has multiple windows to allow your skin to breathe, while still providing the support needed to help you heal.



©2021 BRIJJIT MEDICAL LLC. ALL RIGHTS RESERVED. COVERED BY U.S. PATENT NOS. 10,327,774 AND D862,695. OTHER PATENTS APPLIED FOR IN THE U.S. AND ABROAD. BRIJJIT, "HEAL BEAUTIFULLY," AND THE BRIJJIT HALO LOGO ARE MARKS OF EMRGE, LLC, USED UNDER LICENSE.

Find us



MKC-10011-00